medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T: + 32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Cristovan de Vita, n.º 260, Un. 30,
Centro Logístico Raposo Tavares,
Vila Camargo
Vargem Grande Paulista - SP
CEP 06730-000
Brasil
T: +55-11-3500 8005
sac@medibrasil.com
www.medibrasil.com

MAXIS a.s.,
medi group company
Slezská 2127/13
120 00 Prague 2
Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

medi Danmark ApS
Veilegardsvej 59
2665 Vallensbaek Strand
Denmark
T +45-70 25 56 10
F +45-70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Bayreuth Espana SL
C/Canigo 2-6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34-932 60 04 00
F +34-932 60 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33-1 48 61 76 10
F +33-1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1 371-0090
F +36 1 371-0091
info@medi.hu
www.medi.hu

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31-76 57 22 555
F +31-76 57 22 565
info@medi.nl
www.medi.nl

medi Orient M.E.A. ZCC
TAURUS GROUP
Suite No. M01 Wasel Al Mamzar Bldg.
Al Wuhida Road, Al Mamzar
P.O.BOX NO. 91464
DUBAI, UAE
United Arab Emirates
T: +971-4 2557344
F: +971- 4 2557399
md@taurusgroup.net

medi Polska Sp. z o.o.
Zygmunta Starego 26
44-100 Gliwice
Poland
T +48-32 230 60 21
F +48-32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
Rua do Centro Cultural, no. 43
1700-106 Lisbon
Portugal
T +351-21 843 71 60
F +351-21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova Street 17
117342 Moscow
Russia
T +7-495 374 04 56
F +7-495 374 04 56
info@medirus.ru
www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
Rm 1806, Building 1 Sandhill Plaza
No. 2290 Zuchongzhi Road
Pudong New District
200131 Shanghai
People's Republic of China
T: +86-21 61761988
F: +86-21 61769607
info@medi.cn

Medi Turk Ortopedi Medikal Ic ve Dis Tic.
AS
Kültür mah. çaldıran sok.14/3
06420 Çankaya/Ankara
Turkey
T +90 312 435 20 26
F +90 312 434 22 82
info@medi-turk.com
www.medi-turk.com

medi UK Ltd.
Plough Lane
Hereford HR4 0EL
Great Britain
T +44-1432 37 35 00
F +44-1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

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Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist

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medi 4C flex

Intended purpose

medi 4C flex is a hyperextension brace. The product is exclusively to be used for the orthotic fitting of the lumbar spine / thoracic spine and only on unbroken skin or if any wounds have been properly covered.

Indications

- Stable vertebral compression fractures of the lower thoracic and lumbar spine
- Axial stabilisation for osteoporotic vertebral collapse in the thoracic and lumbar spine
- Medium-term immobilisation after intervertebral disc surgery
- External stabilisation in juvenile Scheuermann's disease
- Temporary orthotic care after open surgery of unstable vertebral body fractures
- Permanent orthotic care for inoperable spinal tumours and metastases
- Postoperative orthotic care after surgery for tumours and metastases of vertebral bodies
- Aftercare of spinal decompression surgery with and without internal fixation

Contraindications

The medi 4C flex hyperextension orthosis should not be used for patients with unstable lumbar or thoracic vertebral fractures with and without neurological deficit which require surgery. Since skin disorders in the pad region may be relative contraindications, it is advisable to consult the attending physician before starting treatment. If there are no dermatological objections to using the orthosis, it can be used as described.

Fitting instructions

Please wear the product on thin underwear.

• Adjusting the width

First loosen the screws on the chassis and adjust the orthosis to the width of the patient's body. Please always check that the functioning of the hinged fastener is not restricted.

Now loosen the two screws on the side of the chassis and the screws on the fastener. Adjust the orthosis to suit the patient's length – distance from the symphysis pubis to the subclavicular contact area. Then retighten all the screws.

• Adjusting the strap length

Using the Velcro fastener and the reversing loop adjust the length of the strap so that the pad comes to lie in the middle of the back. Remember that a few centimetres of the strap will have to be pulled through the fastener.

• Opening and closing the fastener

Slide the grey bar sideways to open the fastener. To close the orthosis flip the cover over until it audibly clicks into place.

Care instructions

The orthosis can be cleaned with a damp cloth, the back pad with a brush.

- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

Aluminium, PVC

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

Your medi Team

Wishes you a quick return to full fitness.

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